

intellijoint HIP[®] FAQ

Surgeons

Do I have to use a certain implant manufacturer with Intellijoint HIP?

You can use any implant manufacturer with Intellijoint HIP. The system is open platform.

Do I need a CT scan to use Intellijoint HIP?

No, Intellijoint HIP is an image-less computer assisted navigation system. The system is registered to the patient through positioning and anatomical landmarks.

Can I use Intellijoint HIP on my revision cases?

Yes. Additionally in revision cases, the primary acetabular cup position can be registered and used as an intraoperative baseline.

Can Intellijoint HIP be used with a Direct Anterior approach?

Yes. The system has been FDA cleared for Direct Anterior approach.

Can I trial the system in the OR?

Yes. Please contact your local Intellijoint Surgical[®] representative to schedule a clinical evaluation.

What information does Intellijoint HIP provide me clinically?

Intellijoint HIP provides real-time, quantitative measurements of acetabular component anteversion and inclination, as well as leg length and offset.

Does Intellijoint HIP offer pre-operative planning?

Intellijoint HIP can deliver patient specific plans from Intellijoint VIEW[™] that includes a functional cup plan, and leg length & offset targets.

Does Intellijoint HIP account for intraoperative movement of the pelvis?

Yes. Our patented core technology tracks the patient's position directly, accounting for any pelvic movement that occurs during surgery.

How much does the system cost?

Intellijoint HIP is sold based on a fee-per-use model. The cost includes the Intellijoint HIP software, disposables, and around-the-clock support. Please contact your local representative for pricing.

Who is my Intellijoint representative?

To connect you with a local representative, please email sales@intellijointsurgical.com or call 1-888-232-2634.

intellijoint HIP[®] FAQ

Hospital Administrators

A surgeon is interested in Intellijoint HIP. What is the cost of the system?

Intellijoint Surgical[®] provides Intellijoint HIP primarily through a fee-per-use model. Capital and operating leases are also available. Please contact your local representative for pricing.

Are there disposables required for the system?

Yes. With the fee-per-use model, the cost includes the technology and the disposables.

How much staff training is required?

Minimal staff training is required as Intellijoint HIP is a surgeon-controlled system. Intellijoint will in-service your OR staff prior to an evaluation.

What does intellijoint HIP provide the surgeon with?

Intellijoint HIP accurately displays the position of the acetabular cup implant and any subsequent leg length and offset changes. Surgeons can use this information to deliver their patient specific surgical plan from Intellijoint VIEW.

Does intellijoint HIP add time to the surgical procedure?

Intellijoint HIP is time neutral and does not significantly increase surgical procedure time. Surgeons become proficient within 5 cases.

Is pre-operative planning included with Intellijoint HIP?

Intellijoint HIP can deliver patient specific surgical plans from Intellijoint VIEW which is included with the use of Intellijoint HIP.

Do I have to purchase Intellijoint Surgical's implants to use the system?

No, surgeons can continue to use their implants of choice. Intellijoint Surgical does not sell implants.

Do you have clinical data to support improved outcomes?

Yes. Please visit www.inteljointsurgical.com or contact your local representative to receive the most up-to-date clinical studies.

Who is my Intellijoint representative?

To connect you with a local representative, please email sales@inteljointsurgical.com or call 1-888-232-2634.